



New Instruments


## Maintenance of Instruments

- Keep instruments open for cleaning!
- Rinse with distilled water!
- Ensure sufficient drying time!



- Intended use!
- Apply the instruments for wet or dry handling!

Operating Room

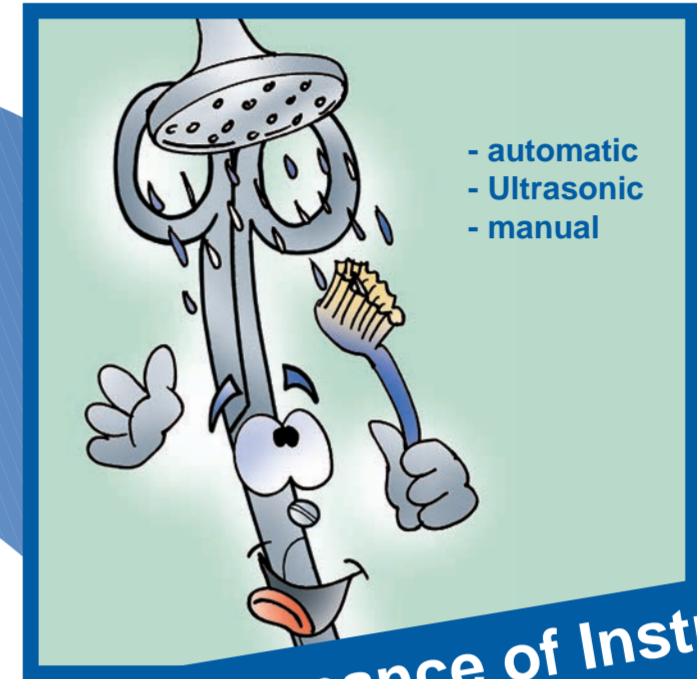

- automatic
- Ultrasonic
- manual

Maintenance of Instruments




Steam sterilization

- Use only distilled water!
- $\geq 134^\circ \text{C}$ !
- Respect norms!

Lawton's  
**Lucky  
Lawrence**


Visual check and functional test

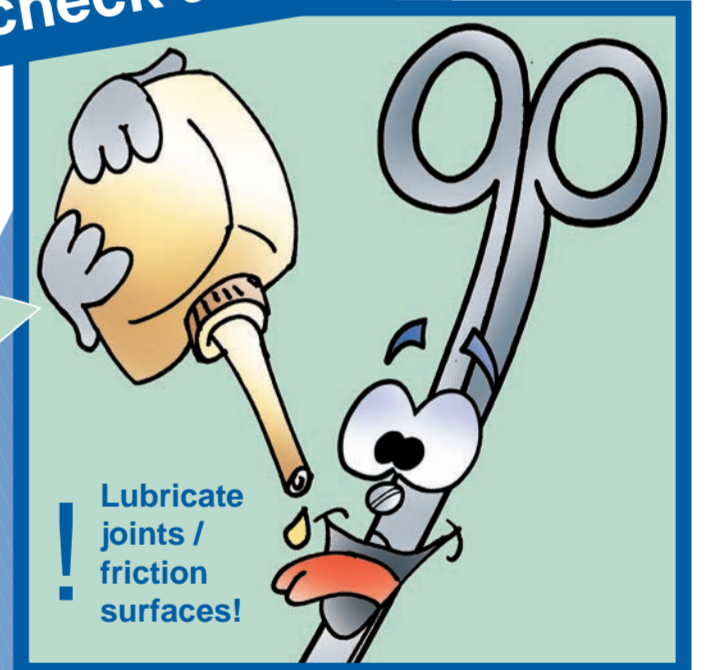

- Handle special instruments separately!

- Lubricate joints / friction surfaces!

Sort and pack

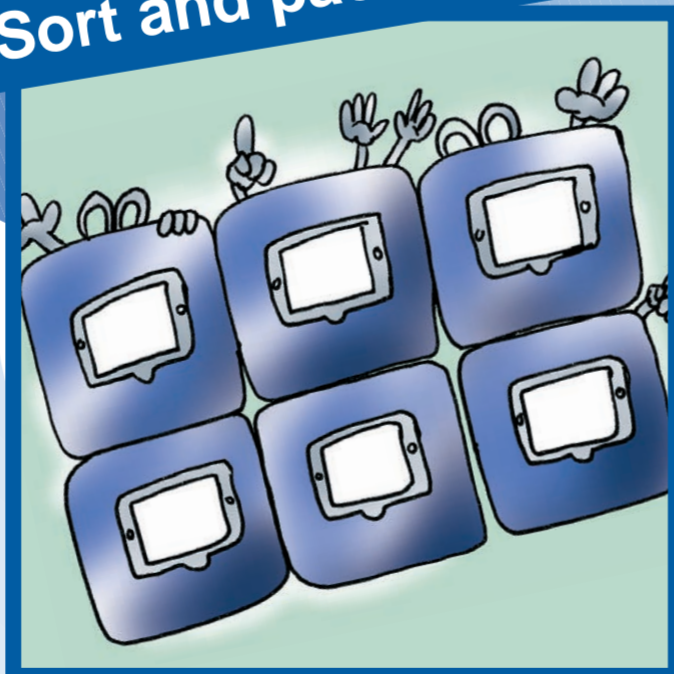

- Open ratchet!
- Don't overload trays!

Defective Instruments



Please order our brief handbook - „Maintenance of Instruments“ for free!

[www.lawton.de](http://www.lawton.de)

Your manufacturer:

LAWTON GmbH & Co. KG  
Württembergische Straße 23  
78567 Fridingen/Germany  
Tel. +49 (0) 7463 9965-0

